

Endocavitary Transducer Workflow

PROVIDERS

Scott Kurpiel MD MBA

Origin: March 2017

Director of Ultrasound

Department of Emergency Medicine

Baylor University Medical Center

Endocavitary Cabinets

- Located end of hall between C31 and C32
 - Door code 1-2-3-4-Enter-Turn Knob
- Techs transport transducers
 - Retrieve from Clean cabinet
 - Return to Dirty cabinet after use
 - Techs stock US machines with probe covers and sealable bags to transport to dirty cabinet after use

Endocavitary Provider Responsibility

1. Initiate process with MH order
 - MH folder: “Ultrasound-endocavitary”
2. Disconnect transducer to free port and connect ECT
3. Place ECT probe cover
4. Enter data, save images, end exam
5. Remove/Dispose ECT probe cover
6. Disconnect ECT and reconnect transducer
7. Give ECT to chaperone/tech

ECT Standard Operating Procedure

1. Initiate process with MH ultrasound folder
 - “Endocavitary transducer to patient room”


📁 Ultrasound-Endocavitary

- 📶 Endocavitary transducer to patient room
- 📶 Place Patient On Pelvic Bed
- 📶 Return Endocavitary transducer to cabinet for high level disinfection after use
- 📶 Ultrasound ED Machine To Bedside




ECT Standard Operating Procedure

2. Disconnect transducer to free port and connect ECT

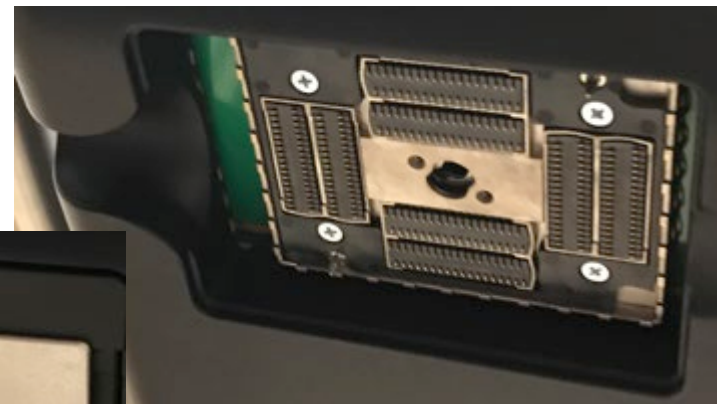

ECT Standard Operating Procedure

3. Place ECT probe cover

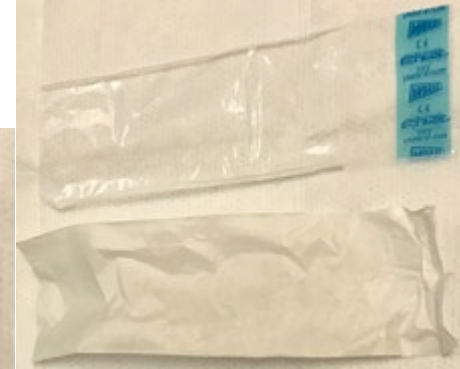

ECT Standard Operating Procedure

4. Enter data, save images, end exam
 - Document indication/findings/interpretation in MH
5. Remove/Dispose ECT probe cover
6. **Disconnect ECT and reconnect transducer**
7. Wipe down visible soilage, give ECT to chaperone/tech

PLEASE:

- *Take ownership over our transducers**
- each costs ~\$6,500**